ekuore

Speculums

Guide of use

1. Símbolos y normativa

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1. Symbology and regulation

Attention. Read the user manual for general precautions and instructions

Serial number

Reference

Manufacturer's Name and Address

Complies with:

EN60601-1:2006+A1:2013+A12:2014
EN60601-1:2015,
EN60950-1:2006+A1:2010+A2:2013

WARNING

- Do not use the product if present any damage.
- Clean the otoscope from patient to patient.
- Check the product box before use, verify that it contains all necessary components.
- The use of accessories other than those indicated by the manufacturer may result in increased emissions or decreased immunity of the product.
- Strictly follow the cleaning instructions to avoid contagion and/or infection.

2. Technical service

Modifications to the product are not permitted. It must be repaired by authorised personnel. For repairs, send equipment to:

eKuore Technical Service

C/Valencia 45B. 46100

Burjassot, Valencia. España

Outside Spain, for information on the maintenance and repair of the product, contact an authorized distributor or send an email to: support@ekuore.com

3. Cleaning and disinfection



Consult otoscope's user manual to know how to clean, disinfect and how to do the maintenance of the product.

How to do an exam

1. Choose the right speculum size


Select the largest possible speculum that your patient's ear will accommodate. When inserted, the speculum should fit snugly into the outer third of the ear canal. Speculums that are too small can cause discomfort and reduce how much of the ear you can examine. Use the following guidelines for speculum size:

1. For adults, 4-6mm
2. For children, 3-4mm

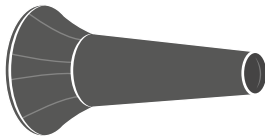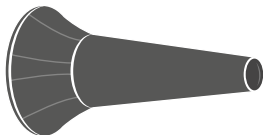1**2**

2. Place the speculum in the otoscope

To place a speculum in an eKuore Pro otoscope you just have to **put it on and press down slightly**, unlike conventional otoscopes, where you must turn the piece also. In this case it is **important do not turn it**, because the device could be damaged.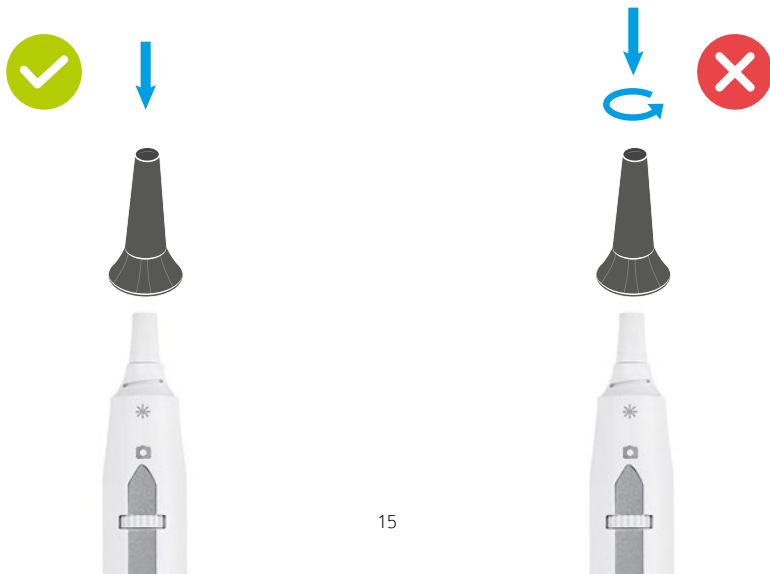



3. Examine the external ear

Without using the otoscope, take a look at the person's external ear and notice any redness, discharge or swelling. Manipulate the ear gently and ask the patient if there is any pain.

4. Insert the otoscope slowly into the ear canal


First turn on the otoscope and connect it, then slowly insert the pointed end of it into the ear canal. Steady your hand on the side of the individual's face if necessary. Slow and gentle insertion can prevent unwanted movement in your patient. It also keeps your hand and scope in line with the ear and minimizes the risk of injury.

Avoid putting too much pressure on the otoscope, which can bump the inner canal wall, causing the patient discomfort.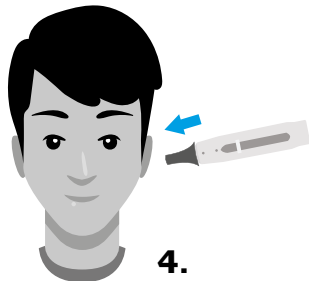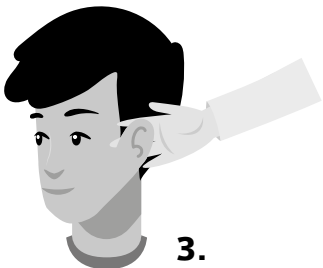

5. Push the speculum 1 to 2 centimeters into the canal

Insert it at most 1 to 2 centimeters and then use the light to view beyond the tip of the speculum. Stop the examination immediately if the patient expresses any pain or discomfort. Examine the middle ear and eardrum.
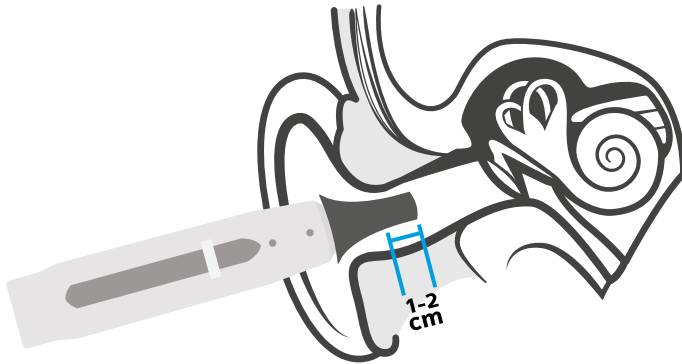


6. Angle the otoscope

Angle the tip of the otoscope towards the person's nose. This follows the normal angle of the ear canal. From here, move the otoscope gently at different angles. This allows you to view the person's eardrum and canal walls. Stop the exam at any sign of increased pain or discomfort.

7. Remove the otoscope


Return the otoscope back to your starting position. Gently take the speculum and scope out of the person's ear canal and outer ear.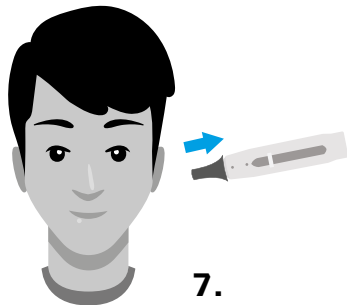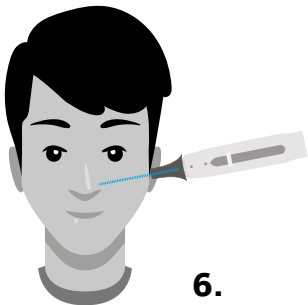

Throw away the speculum

Remove the speculum from the otoscope. Throw it away in a certified medical waste container to minimize the spread of disease or infection to other patients.

If you don't have disposable speculums, scrub each tip with hot water to remove excess wax. Then soak the speculum in a covered dish of rubbing alcohol for 10 minutes.

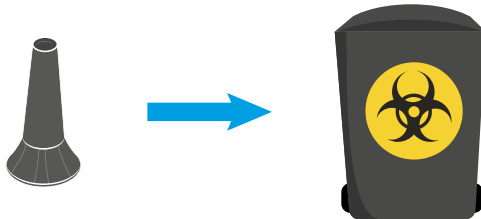GET MORE SPECULUM

Speculums you will find in the packaging are single-use, you can get more calling to [+34 961 15 67 38](tel:+34961156738) or entering on our online store www.ekuore.com/shop

ekuore

Speculums

REF Y008

www.ekuore.com

info@ekuore.com

+34 961 156 738